Blood sample collection kit

Test Kit Instructions

Instructions du Kit de Test

Testkit Anweisungen

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Healthy Stuff Online Ltd.
Unit 1A Bonington Complex,
Trent Lane, Trent Lane Industrial
Estate, Derby, DE74 2PY, UK.
www.healthystuff.com
info@healthystuff.com

EC REP

Obelis, Bd Général Wahis,
53B-1030 Brussels, Belgium.
Tel +32 27325954
Fax +32 27326003

Index of Symbols

IVD	In vitro diagnostics	IVD	Diagnostics in vitro	IVD	In-vitro Diagnostik	IVD	Diagnósticos in vitro	IVD	In-vitro diagnostiek
	Read the instruction before taking the sample		Lire les instructions avant de prendre l'échantillon		Lesen Sie die Anweisung, bevor Sie die Probe entnehmen		Lea las instrucciones antes de tomar la muestra		Lees de instructies alvorens het bloedmonster af te nemen
	For single use only		Pour un seul usage uniquement		Nur zum einmaligen Gebrauch		Solo para uso individual		Uitsluitend voor eenmalig gebruik
	Store at a temperature between 2°C and 30°C		Conserver à une température entre 2°C et 30°C		Bei einer Temperatur zwischen 2°C und 30°C lagern		Almacenar a una temperatura de entre 2°C y 30°C		Bewaar bij een temperatuur tussen de 2°C en 30°C
	Contains 1 kit		Contient 1 kit		Enthält 1 Testkit		Contiene 1 kit		Bevat 1 kit
	Manufacturer		Fabricant		Hersteller		Fabricante		Producent
	Expiration Date		Date de Fabrication		Ablaufdatum		Fecha de caducidad		Vervalidatum
	Batch Number		Numéro de Lot		Chargenummer		Número de lote		Batchnummer
	REF Order Ref		N° de Commande		Bestellreferenz		Referencia del pedido		Bestellingsreferentie

EN | IMPORTANT INFORMATION

For best results, we recommend collecting your sample on **Monday to Tuesday** – and post it on the same day it was collected (within 24 hours; please consider any public holidays).

Blood Sample Collection Kit:

Intended Use:

A kit for the collection of whole blood using a lancet and a dried blood spot card. For professional in vitro diagnostic use only.

Summary:

A blood test (dried blood) for an in-vitro-immunoassay for the simultaneous measurement of human-specific IgE and/or IgG4 antibodies against specific allergens.

Materials Provided:

Alcohol Wipe, Plaster, Gauze/Absorbent Pad, Lancets, Blood Collection Card

GENERAL INFORMATION

- Please read these instructions carefully before you begin
- Please don't take your blood sample immediately after strenuous exercise
- If you have any difficulties taking your blood sample (or if the kit is incomplete), please contact our Customer Service Department. They always treat such matters confidentially
- Please note: We require at least 4 full circles to the outer edge to enable us to complete your testing.

CONTAGIOUS ILLNESSES

Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

WARNINGS

- Store this kit out of reach of children
- This kit is for the use of people aged over 2 years; under 16s should be supervised by an adult
- If you're pregnant or breastfeeding, please consult your doctor before using this kit
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, please consult your doctor before using this kit
- The included lancets are exclusively for your own use

If you have any questions, please send us an email at info@healthystuff.com

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